

VaccinePorter®

User Guide

Keeping medicines effective

VaccinePorter® Carriers are validated to transport vaccines and other heat sensitive products in safety, maintaining thermal performance and integrity throughout its journey



Prior to use:

- **VaccinePorter®** — Pre-condition Medicool® packs at **+5°C** for a minimum of 24 hours
- **Vaccine MiniPorter™** — Freeze Medicool® packs at **-18°C** for a minimum of 24 hours. Remove 20 mins before use

System assembly and packing

1. Open bag and inner chamber lid
2. Place pre-conditioned Medicool® packs on the base of the system
3. Place the product to be transported in the centre of the VaccinePorter®
4. Place Medicool® on each side of the system in accordance with instructions for use, surrounding the product load
5. Place Medicool® on top of the product
6. Place the lid correctly and seal protective outer bag immediately



VaccinePorter® carriers are validated so that product can be added and removed during use. However please do not open unnecessarily and remember to re-seal the system immediately.

To re-use the system:

- Conduct a weight check and visual squeeze to ensure there is no loss of liquid. If the Medicool® is $\pm 20g$ different to the weight shown then we recommend immediate disposal and replacement
- Chipped or broken inner chambers need to be replaced before use as this can affect the thermal performance



Replacement Medicool® packs and inner chambers are available to purchase separately from Helapet.

Further information is available from:

Helapet

A CliniMed® Group Company

Helapet Limited, Lister House, Blackburn Road, Houghton Regis, Bedfordshire. LU5 5BQ
Telephone 0800 0328 428 Fax +44 (0) 1582 501981
E-mail: sales@helapet.co.uk | Web: helapet.co.uk

