




Prior to use:

Before inserting each Medicool® into the VaccinePorter® system, please follow these instructions:

- Refrigerate 6 x Medicool® 28 at +5°C, ± 3°C for a **minimum** of 24 hours*.
- Medicool® cool packs should be stacked flat on storage shelves no more than 3 high, with 50mm of free air circulation around the packs.

Instructions for Use - Please Turn Over

*Due to the potential variation in fridge performance, frequency of opening and other fridge contents, it is recommended that Medicool® pre-conditioning is validated by the user.

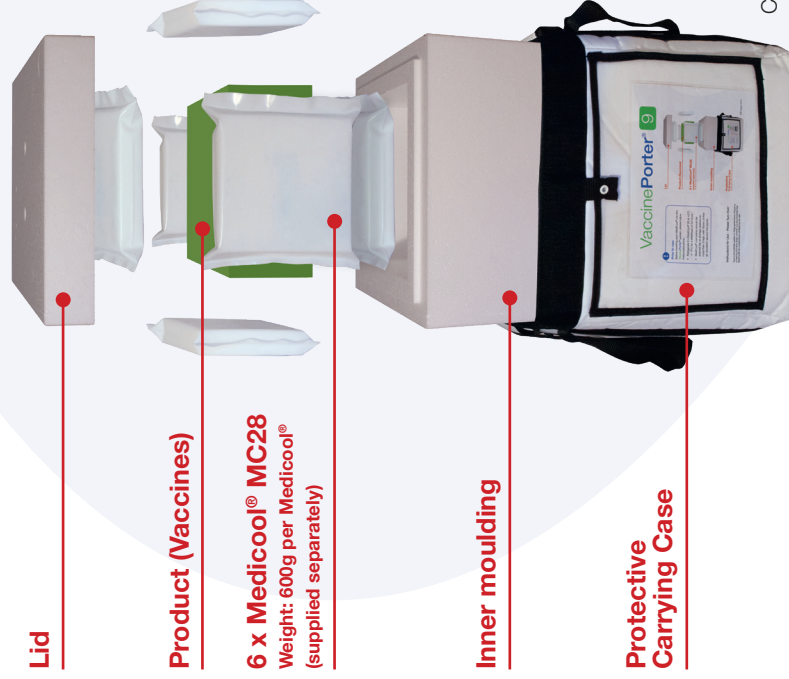

CP1422/1017

Instructions for Use

System assembly and product packing

Pack and seal the VaccinePorter® system swiftly, as follows:

1. Place 1 x Medicool® 28 on the base of the system.
2. Place the product to be transported in the centre of the VaccinePorter®.
3. Place 1 x Medicool® 28 on each side of the system, surrounding the product load.
4. Place 1 x Medicool® 28 on top of the product.
5. Close the lid and ensure it is sealed by closing the outer carrying case immediately.
6. Only open the VaccinePorter® if necessary and remember to re-seal as soon as possible.

Although the VaccinePorter® has been validated so that the product can be added and removed from the sealed system during use, do not open unnecessarily and remember to re-seal the system quickly.

To re-use the system:

In case you decide to re-use the system, please follow these steps below prior to conditioning the Medicool® cool packs:

- A weight check and visual squeeze check will need to be carried out to ensure there is no loss of liquid. If the Medicool® is ±20g different to the weight shown on the Medicool® (overleaf) or shows any signs of leaking, then it is recommended that it is disposed of.
- We recommend that any chipped or broken polystyrene inners are replaced before use as this can affect the thermal performance of the VaccinePorter®.

Further information is available from:



Helapet Limited, Lister House, Blackburn Road, Houghton Regis, Bedfordshire. LU5 5BQ
Telephone 0800 0328 428 Fax +44 (0) 1582 501981

E-mail: sales@helapet.co.uk | Web: helapet.co.uk